

SYNAPSE MOBILITY CHEAT SHEET



Keyboard Shortcut Guide for Synapse Mobility Web

Global Functions

Key or Interaction	Function
HOME	Reset view
-	Zoom out
=	Zoom in
ALT + mouse wheel	Zoom in/out in MIP/MPR and 2D Rotate about horizontal axis in 3D
SHIFT + left-click + drag	Pan/Translate
DELETE	Remove selected mark-up Removes selected area in scalpel mode
Right-click + drag	Window/level
V	Reset window/level

MIP/MPR Functions

Key or Interaction	Function
E	Toggle extents
D	Reset all views
CTRL + click	Triangulate
S	Toggle slabbing
T	Disables slabbing when pressed
C	Resets the view currently under the mouse cursor
ALT + click + drag	Auto-scroll through slices (in MIP/MPR and 2D)

2D Functions

Key or Interaction	Function
D	Reset pan
S	Reset view to its default slice
1 - 0	Window/level presets
Alt + click + drag	Auto scroll through slices
I	Invert intensities
O	Toggle magnifying glass

3D Functions

Key or Interaction	Function
R	View volume from Right
L	View volume from Left
S	View volume from Superior
I	View volume from Inferior
A	View volume from Anterior
P	View volume from Posterior
G	Reset clipping planes
O	Toggle Lens Tool
C	Toggle clipping planes
Z	Undo scalpel (can be used repeatedly)